

### **Setting verification mode for Roger Touchscreen with firmware version 3.0 onwards**

1. Turn on the transmitter.
2. Tap on Settings.
3. Tap on Advanced.
4. Tap on Verification Mode.
5. Tap on Activate.
6. Press the return arrow then press it again on the next screen.
7. Press the Home button.
8. The transmitter is now in Verification mode.
9. Be aware that the transmitter may exit verification mode after a period of time but will still display Verification mode.
10. For this reason, it is best to activate Verification mode immediately before carrying out the procedure.